

# Retro-Fit Manual Diamond 2 Outlet



# Important Note

Due to unknown factors of the existing equipment, projects requiring Retro-Fit Conversions must be fully investigated by the installer.

It is the responsibility of the installer to determine if the Retro-Fit will function correctly and if additional components are required. Amico is not responsible for any additional components required.

## Retro-Fit Instructions

**Note:** This application is for a complete conversion.

### Ohmeda Diamond 2 & 3 Outlet

Remove screws to expose rough-in.



Remove all components from existing rough-in.



Insert sleeve with o-ring and other components as seen below.



# Retro-Fit Instructions

## Amico Diamond 2 Conversion Kit.



Retaining Ring



Washer



Seat



Check Cap



Spring



Sleeve with O Ring

Place the chrome 4.5" frame over the rough-in and screw in the latch.



A completed Diamond 2 assembly conversion from Ohmeda to Amico.

### WARNING

Ohmeda issued a Mandatory Field modification for its back body on Diamond 3 Outlets manufactured between 1990 and 1993. If this Ohmeda field modification was not applied, the Outlet MUST NOT be Retrofitted since this could cause total loss of gas flow through the Outlet.



# www.amico.com

Amico Pipeline | [www.amico.com](http://www.amico.com)

85 Fulton Way, Richmond Hill  
Ontario, L4B 2N4, Canada

71 East Industry Court, Deer Park  
NY 11729, U.S.A

Toll Free Tel: 1.877.462.6426

Toll Free Fax: 1.866.440.4986

Tel: 905.764.0800

Fax: 905.764.0862

Email: [info@amico.com](mailto:info@amico.com)